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COLDISES[®] Nasenöl-Spray

Was ist COLDISES Nasenöl-Spray?

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COLDISES[®] Nasal Oil-Spray

What is COLDISES Nasal Oil-Spray?

COLDISES Nasal Oil-Spray is used for the prophylactic and supportive treatment of a dry or incrustated nasal mucosa. It is sprayed into the nose for moistening the nasal mucous membrane.

What does COLDISES Nasal Oil-Spray contain?

COLDISES Nasal Oil-Spray contains sesame oil, furthermore isopropylmyristate, vitamin E, bergamot oil, clove oil and an antioxidant (BHT).

How does COLDISES Nasal Oil-Spray work?

Due to the high-quality ingredients COLDISES has an abirritant, regenerative and soothing effect on the nasal mucous membrane. In many cases, it will help to regenerate affected regions and protect from renewed irritation.

Sesame oil is rich in unsaturated fatty acids and has no significant allergic potential.

Vitamin E protects the skin cells against free radicals and oxidative stress. Together, they contribute to the regeneration and care of the nasal mucous membrane keeping it moist and supple.

Therapeutic Indications – What is COLDISES Nasal Oil-Spray used for?

COLDISES Nasal Oil-Spray is used for the prophylactic and supportive treatment of a dry or incrustated nasal mucous membrane in the case of:

- Irritated and dry nasal mucous membrane (caused by dry air or pollution, for example)
- A cold and runny nose
- Allergic cold
- Incrustations

as well as for supporting treatment when you are using decongestant medication.

COLDISES Nasal Oil-Spray can be used in pregnancy and lactation without hesitation.

It is suitable for use in adults and in children of 3 years and older.

Dosage and method of administration

– How and how often should you use COLDISES Nasal Oil-Spray?

Several times a day, depending on your individual requirements, spray a few times into the nose.

The following dosage recommendation has been defined: 2 – 3 times daily 1 – 3 puffs into each nostril.

Keep the bottle upright while spraying!

COLDISES Nasal Oil Spray can be used over a longer period of time (up to 30 days).

When should COLDISES Nasal Oil-Spray not be used?

If you are allergic (hypersensitive) to any of the ingredients of COLDISES Nasal Oil-Spray.

Possible side effects/interactions

There are no known side effects up to now.

There have been no studies on interactions with other products.

Precautions for safe use

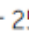
COLDISES Nasal Oil-Spray is a medical device. If your condition does not improve or pain occurs consult a doctor.

For hygienic reasons and to avoid transmission of pathogens, one bottle should be used **by one and the same person** only.

Always keep bottle closed and out of the reach of children!

Pack size: Bottle contains 10 ml with nebuliser

Shelf life: See the expiry date on the package. Do not use longer than 6 weeks after first use.

Storage conditions:  8°C  25°C Store between 8 °C and 25 °C.  Keep away from light.

 **Manufacturer:** MoNo chem-pharm Produkte GmbH, Leystraße 129, A-1200 Vienna - Austria

Distribution: SIGMAPHARM Arzneimittel GmbH, A-1200 Vienna - Austria

Revision date: March 2021