# COLDAMARIS Nasen- und Rachenspray

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STERILE A

# COLDAMARIS Nasal and Throat Spray

## What is COLDAMARIS Nasal and Throat Spray?

COLDAMARIS Nasal and Throat spray is a sterile, isotonic sea salt solution for moistening the nasal and pharyngeal mucosae.

## What does COLDAMARIS Nasal and Throat Spray contain?

COLDAMARIS Nasal and Throat Spray contains standardised sea salt consisting of sodium, potassium, calcium, magnesium, chloride, sulphate, bromide, iodide and bicarbonate ions as well as tyloxapol (wetting agent), citric acid and purified water.

COLDAMARIS Nasal and Throat Spray is free of preservatives.

## How does COLDAMARIS Nasal and Throat Spray work?

As we breath, we inhale many particles and pathogens into the nose and throat. The nasal and pharyngeal mucosae can only fulfil their natural function as a cleansing and protective barrier against external influences if there is sufficient moisture.

Dry indoor air, dirt and dust in ambient air and use of decongestant nasal sprays irritate the nasal mucosa and dry it out. Symptoms of a dry and irritated nasal mucosa may include itching, burning, sneezing or crust formation.

A dry, tickly throat, the need to clear the throat or a hoarse voice can often be the typical initial indication of a cold but can also be caused by an irritation of the throat mucosa due to long periods of talking, singing, smoking or staying in heated or air-conditioned rooms.

COLDAMARIS Nasal and Throat Spray moistens the nasal and pharyngeal mucosae and relieves the symptoms of dryness. The regular moistening supports the natural function of the mucosae.

## What is COLDAMARIS Nasal and Throat Spray used for?

- For moisturising in the case of dry nasal and/or pharyngeal mucosae.
- To facilitate nasal cleansing in the case of encrusted mucosa and increased exposure to dust, smoke, pollen.
- For dissolving mucus in case of a blocked nose.

COLDAMARIS Nasal and Throat Spray is suitable for use in children of 1 year and older and in adults.

In infants under 1 year of age, COLDAMARIS should only be used when recommended by a physician.

COLDAMARIS can be used during pregnancy and when breastfeeding without any concerns.

## How and how often should you use COLDAMARIS Nasal and Throat Spray?

Spray one or more puffs into each nostril or the throat several times a day, as required.

The application may be repeated as often as desired and may be continued for a longer period of time without intervals between use.

Remove the protective cap before use and pump a few times if necessary until a uniform spray mist is emitted. COLDAMARIS Nasal and Throat Spray is now ready for use.

Keep the bottle as upright as possible when spraying. The suction tube must protrude into the liquid. Insert the spray nozzle carefully and not too deeply to avoid injury.

If the functionality of the spray nozzle is impaired, press the nozzle several times in an upright position until the spray mist is emitted again.

After use, put the protective cap back on the nozzle.

## When should you not use COLDAMARIS Nasal and Throat Spray?

If you are hypersensitive (allergic) to one of the ingredients of the product.

## Possible side effects/interactions

There are no known side effects to date.

There have been no studies on interactions with other products.

## Precautions for safe use

Seek medical advice if your symptoms do not improve or if you experience pain. In case of heavy crust formation or inflammatory processes, make sure to consult a physician.

For reasons of hygiene and to avoid transmission of pathogens, each spray bottle should be used **by one and the same person only**.

## Keep out of reach of children!

**Pack size:** Bottle with 20 ml sterile solution, with spray attachment. 1 puff = 0.14 ml.

**Shelf life:** Because of its special filter system the solution remains sterile even with repeated use. COLDAMARIS Nasal and Throat Spray can therefore be used until the expiry date mentioned on the package. The expiry date refers to the last day of that month.

**Storage instructions:**  Do not store above 25 °C.  Keep away from sunlight.

**Manufacturer:** MoNo chem-pharm Produkte GmbH, Leystraße 129, 1200 Vienna - Austria

**Distribution:** SIGMAPHARM Arzneimittel GmbH, 1200 Vienna - Austria

Revision date: June 2022

CE 0123

STERILE A